

For technical support call Massage-Tools.com at 866-928-9955 x2 (Product Questions).


Quick Assembly Guide - Silver Fox 1003 Infrared Lamp and Stand

Included Parts for Stand – 5 small nuts with locking washers, metal locking plate, single large nut with washer and locking washer. (*wrench included*)



1. Assemble Lamp Floor Stand

- a. Locate the 5 metal legs and secure to metal locking plate using 5 metal nuts and locking washers provided. (*see picture A below*)
- b. Insert leg assembly into plastic caster base. (*see picture B below*)

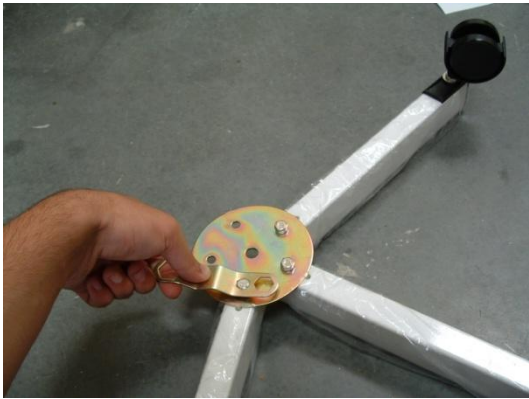

Picture A



Picture B

- c. Insert pole into caster base and secure from opposite side using single large nut with washer and locking washer. (*see picture C below*)



Picture C

2. Insert Lamp into Stand